**biber**

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**Für Schweiz**

Swissmedic, Swiss Agency for Therapeutic Products  
Medical Devices Division  
Hallerstrasse 7, 3012 Bern, Switzerland

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**Back support belt for housework and gardening 458450**

Version: 2503EL\_2003\_Sz

**Back support belt for housework and gardening XXL 458451**

Version: 2703EL\_2102\_Sz

**Scope of delivery**

**1 x Back support belt for housework and gardening**

Read the instructions for use completely and carefully!

**Purpose**

To activate and support the muscles of the abdomen and back through external pressure (compression). To support the spine and promote an upright posture.

- slipped discs
- sciatica
- lumbago
- pain from overstraining

For personal use

**Contraindications / Unsuitability**

The belt should only be used after consultation with your doctor in the case of the following pathologies:

- sensory disturbances and movement disorders, e.g. with diabetes
- open wounds or internal injuries
- allergic reactions to the belt
- on small children or people who cannot call for attention if the belt is uncomfortably tight.

**Instructions for safe use of the product:**

There are no known side effects that involve the entire organism. Proper use is a prerequisite. Product liability is excluded in the case of improper use.

- If the belt is too tight or feels uncomfortable, remove it immediately.
- Wear the belt only for an hour at a time, not permanently.
- Consult your doctor immediately if the symptoms worsen or you notice any changes in your condition.

Do not use:

- while sleeping

**Use**

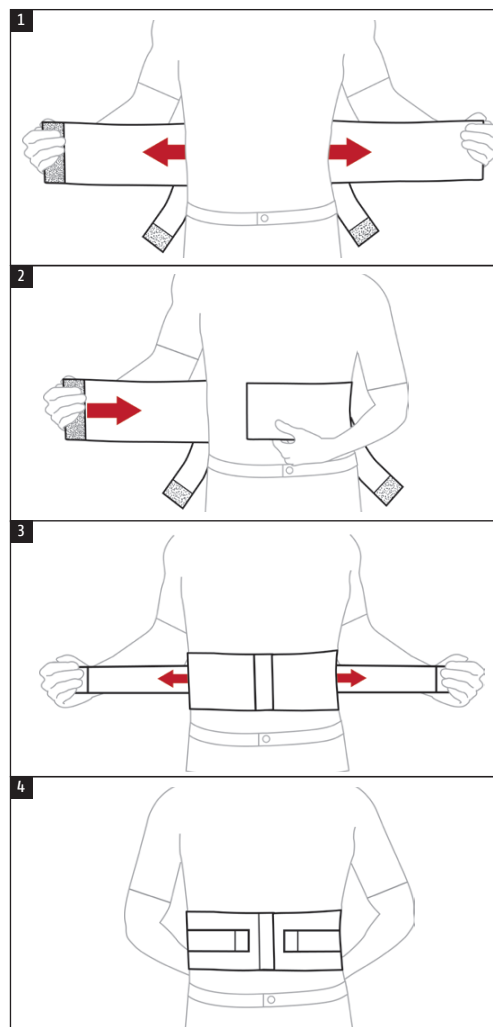
Two elasticated straps in the back area and additional pull straps for individual adjustment ensure extra support.

**Cleaning**

Close the Velcro fasteners before washing to prevent damage to other items of laundry. The belt can be washed with the household laundry – see the care instructions.

**Putting on the back support belt**

1. Undo the Velcro fasteners of the two additional straps of the belt. Hold the support belt with the Velcro fastener (inside) in your right hand and pass it around your back. First pull the two straps way from the body at the side.
2. First fold in the left side, then the right. Then use light pressure to attach the Velcro fastener to any point on the belt.
- 3./4. Proceed in exactly the same way to fasten the two additional straps.

All serious incidents involving the product must be reported to the manufacturer and the responsible authorities of the Member State where the user and/or patient is located.

**For Germany**

Federal Institute for Drugs and Medical Devices  
 Medical Devices Division  
 Kurt-Georg-Kiesinger-Allee 3  
 53175 Bonn  
<https://www2.bfarm.de/medprod/mpsv/>

**For Austria**

Federal Ministry of Health  
 Dept. Pharmaceuticals and Medical Devices  
 Radetzkystrasse 2, 1030 Vienna  
 Email: [meddev@bmgf.gv.at](mailto:meddev@bmgf.gv.at)

**For Switzerland**

Swissmedic, Swiss Agency for  
 Therapeutic Products  
 Medical Devices Division  
 Hallerstrasse 7, 3012 Bern, Switzerland

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Biber Umweltprodukte Versand GmbH
Hydas GmbH & Co. KG, Hirzenhainer Straße 3,